APLICAÇÃO ARCHITECT

Apo B FS

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GENERAL PARAMETERS					
Name	Apo B		Assay type	Photometric	
Assay number			Assay availability	Enabled	
Assay version			Cal version		
REACTION DEFINITION					
Reaction mode	End up		Main read time	31 – 33	
Primary wavelength	340		Flex read time		
Secondary wavelength			Color correction read time	0 – 0	
Last read required	33		Blank read time	0 – 0	
Absorbance range	0.000 – 0.000				
Sample blank type	Self				
Blank assay					
REAGENT/SAMPLE					
Reagent	Apo B				
R1 reagent volume	250		R2 reagent volume	50	
R1 water volume	0		R2 water volume	0	
R1 dispense mode	Type 0		R2 dispense mode	Type 0	
Diluent name	Salina		Diluent dispense mode	Type 0	
Diluent name	Sample volume	Diluted S. Vol.	Diluent volume	Water volume	Dilution factor
STANDARD	2.5	0.0	0	0	1:1
DIL1	0	0	0	0	1:20
VALIDITY CHECKS					
Reaction check type	None				
Read time A range			Read time B range		
Calculation limit			Minimum absorbance		
			Rate linearity %		
Maximum abs. variation	0.000				
CALIBRATION PARAMETERS					
Calibration method	Logit-4				
Use cal factor from			Factor	1.0	
Full interval hours	720		Adjustment interval hours		
Expected cal factor	0		Adjustment level		
Expected cal factor tol. %	0		Default ordering type	Full	
			Blank absorbance rate	0.00 – 0.00	
Span			Span absorbance rate	0 – 0	
Maximum curve fit	0				
Calibrator set name	TruCal U		Replicates	3	
Cal level	Concentration	Sample volume	Diluted S. vol.	Diluent volume	Water volume
Blank	0	2.5	0	0	0
Std 1	*	2.5	0	0	0
Std 2	*	2.5	0	0	0
Std 3	*	2.5	0	0	0
Std 4	*	2.5	0	0	0
SMART WASH					
Component	Reagent/Assay	Wash	Volume	Replicates	Wash protocol
Reagent probe (R1)	All	Water	345	1	
Reagent probe (R2)	All	Water	345	1	
RESULT PARAMETERS					
Linearity range					
Flag range specifications					
RESULT UNITS					
Result concentration units	mg/dL		Correlation factor	1.00	
Result decimal places	1		Intercept	0.00	

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Produzido por: Diasys Diagnostics Systems GmbH