APLICAÇÃO ARCHITECT

Colesterol FS

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GENERAL PARAMETERS					
Name	COL	Assay type	Photometric		
Assay number		Assay availability	Enabled		
Assay version		Cal version			
REACTION DEFINITION					
Reaction mode	End Up	Main read time	31 - 33		
Primary wavelength	500	Flex read time	0 - 0		
Secondary wavelength	660	Color correction read time	0 - 0		
Last read required	33	Blank read time	0 - 0		
Absorbance range	0.000 – 0.000				
Sample blank type	-				
Blank assay					
REAGENT/SAMPLE					
Reagent	COL				
R1 reagent volume	200	R2 reagent volume	0		
R1 water volume	0	R2 water volume	0		
R1 dispense mode	Type 0	R2 dispense mode	Type 0		
Diluent name	Salina	Diluent dispense mode	Type 0		
Diluent name	Sample volume	Diluted S. Vol.	Diluent volume	Water volume	Dilution factor
STANDARD	2.0	0.0	0	0	1:1
DIL1	5.0	10.0	95	0	1:10
DIL2	5.0	4.0	95	0	1:25
VALIDITY CHECKS					
Reaction check type	None				
Read time A range		Read time B range			
Calculation limit		Minimum absorbance			
		Rate linearity %			
Maximum abs. variation	0.000				
CALIBRATION PARAMETERS					
Calibration method	Linear				
Use cal factor from		Factor	1.0		
Full interval hours	720	Adjustment interval hours			
Expected cal factor	0	Adjustment level			
Expected cal factor tol. %	0	Default ordering type	Full		
		Blank absorbance rate	0.00 – 0.00		
Span		Span absorbance rate	0 – 0		
Maximum curve fit	0				
Calibrator set name	TruCal U	Replicates	3		
Cal level	Concentration	Sample volume	Diluted S. vol.	Diluent volume	Water volume
Blank	0	2.0	0	0	0
Std 1	*	2.0	0	0	0
SMART WASH					
Component	Reagent/Assay	Wash	Volume	Replicates	Wash protocol
Reagent probe (R1)	All	Water	345	1	
Reagent probe (R2)	All	Water	345	1	
RESULT PARAMETERS					
Linearity range					
Flag range specifications					
RESULT UNITS					
Result concentration units	mg/dl	Correlation factor	1.00		
Result decimal places	1	Intercept	0.00		

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Produzido por: Diasys Diagnostics Systems GmbH